

TraumaMan Tribune

In This Issue

[Replaceable Tissue Sets](#)

[Helpful Hint #4](#)

[3 System Package Price](#)

[DVD and Carrying Case](#)

Past Issues

[TraumaMan Tribune Vol 1](#)

[TraumaMan Tribune Vol 2](#)


[TraumaMan Tribune Vol 3](#)

[Join Our Mailing List!](#)

Welcome to Volume Four of the TraumaMan Tribune. This issue contains an overview of TraumaMan's replaceable tissue sets, a helpful hint, and details about our new Three System Package!

Confused About Replaceable Tissue Sets?

- Neck Tissue: 2 students
- Chest Tissue: 4 students
- Ab Tissue: 2 students
- Ankle Tissue: 2 students

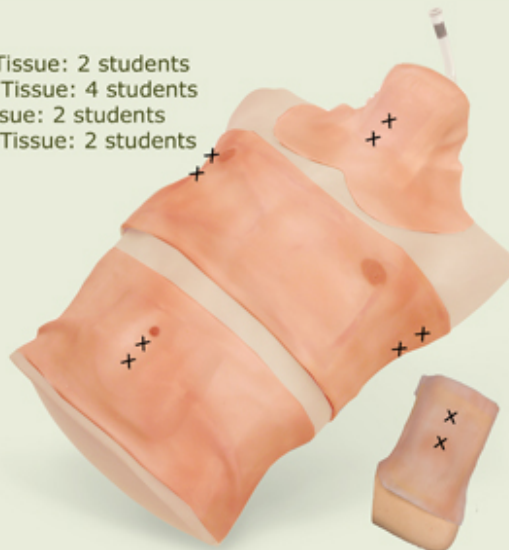

Simulab's replaceable tissue sets are designed for use by multiple students. As shown in the graphic above - the neck, abdominal, and ankle tissues can be used for two student procedures each and the chest tissue can be used for four procedures.

For your convenience, replaceable tissue sets are sold in terms of students. So if you ordered tissues for a Surgical Skills Practicum of 16 students, you would receive eight each of the neck, abdominal, and ankle tissues (two student uses per tissue); and four chest tissues (four student uses per tissue). If your course does not require all four tissues, we can customize the tissues you receive according to your needs.

Still confused? Contact our Customer Care Team at 206-297-1260 or service@simulab.com for more information!

Helpful Hint #4: Successfully performing the Pericardiocentesis




When performing the Pericardiocentesis, it is important to first confirm that your pericardiocentesis kit includes a needle that is at least 3.5 inches long. A needle shorter than this will not be long enough to reach the simulated pericardial fluid.

If you have confirmed that you are using the correct length of needle and you are still not able to complete the procedure, please refer try one of Simulab's troubleshooting tips, such as ensuring the needle is not clogged. These tips can be found in the User's Guide Manual on page 47 or in the 9th section of the User's Guide DVD.

NEW! Three System Package Pricing

Simulab is pleased to offer special pricing when you purchase our Three System Package. Purchase [TraumaMan System](#), [CentraLineMan System](#), and [FemoraLineMan System](#) and save!

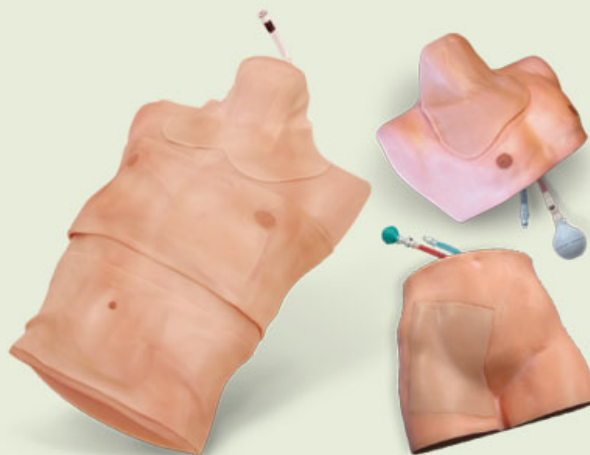

Contact Simulab's Customer Care Team today to learn more.